

Dexis - Configuration

355 [TDO KB](#) August 11, 2026 [Radiography](#) 15.26K

Overview

Dexis integrates with TDO using a bridge. Dexis can be launched through TDO using an automation process.

Detailed Instructions

Go to **Setup tab > Site Configuration > Edit Control Table**.

Click the **Workstation Settings** tab

Under **Radiography Software**, select *Dexis*

For the **Radiography path** change the path to the local installation of Dexis or Dexis 10.855, and is **C:\Dexis**

C:\Dexis - Dexis will not integrate if installed anywhere else, it needs to be **C:\Dexis**

Close the Control Table

To Test

Open **Acquire** tab, click **XRay (tooth icon)** which will launch Dexis outside of TDO.

Acquire your x-ray as you normally would in Dexis.

When the X-ray is up on screen, copy it to your clipboard by hitting **Ctrl + C** on your keyboard.

Go back to the **Acquire** window and click the **Paste Image from Clipboard (clipboard icon)** button, which will paste the X-ray into TDO.

Then, choose the proper image category and click the **Save** button.

This has been tested and verified to work in version 9.3.0 of the Dexis software but may not work in more recent versions of Dexis. Also, as a note, Dexis must first be installed and working on the computer before you can configure in TDO.

Online URL: <https://kb.tdo4endo.com/article.php?id=355>