

Changing Brightness and Contrast of Saved X-Rays

418 [TDO KB](#) November 20, 2025 [Draw](#), [Radiography](#) 9.14K

Overview

TDO allows you to change the brightness and contrast of your images from the pop-up Draw window.

NOTE: The video below was taken using TDO v11. If you are using TDO v12, please note that some of the appearances may seem outdated. The directions and usage, however, are still the same.

Detailed Instructions

For those of you using v12 of TDO, there are some slight changes to the user interface. Rather than sliding across the image itself, the button to change brightness and contrast will now show sliders that you can move back and forth to make adjustments to your image.

The options to adjust your image are now labeled Intensity, Contrast, and Gamma.

Online URL: <https://kb.tdo4endo.com/article.php?id=418>