

How to Reset Organizer to Default Settings

716 [TDO KB](#) February 26, 2026 [Organizer](#), [Organizer](#) 6.55K

Overview

There is a new way in TDO to reset the Image Organizer to its original state without the need to do it manually.


If Organizer is set to display images with specific settings, such as Pane view, sorted by ‘*Taken Date*’, etc., you can hold the **SHIFT** key on the keyboard. This will reset Organizer to its default settings.

Organizer Default settings:


- View: Thumbnails
- Size: 200x200
- Sort: Tooth

Detailed Instructions

Let’s take, for example, that Organizer is set to display images with *Pane* view, sorted by ‘*Taken Date*’, and image size *400x400*

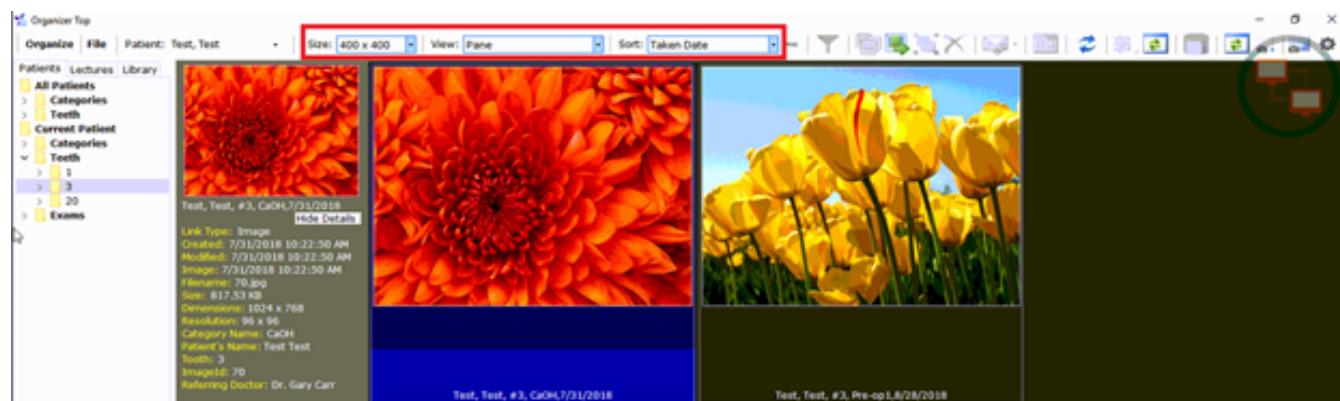

- Close Organizer
- Hold the “**SHIFT**” Key on the keyboard
- Click on the “**Organizer**” button
- Organizer is now set to display its default settings

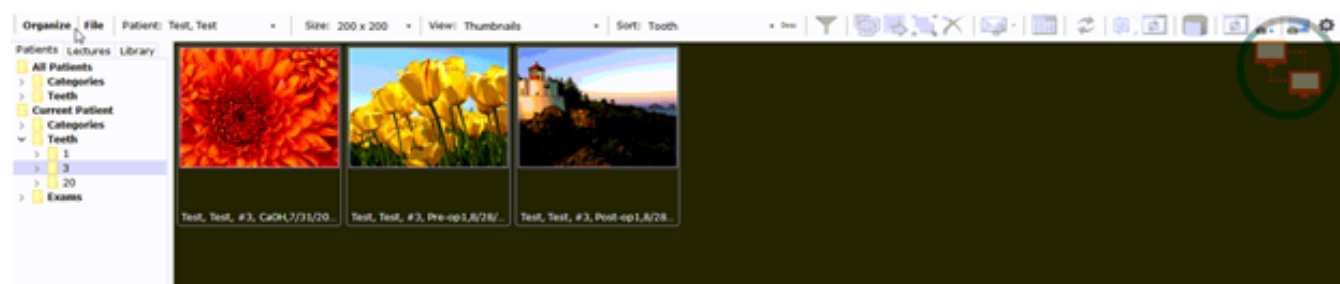

Online URL: <https://kb.tdo4endo.com/article.php?id=716>