

Carestream 1200 and 1500 Intraoral Camera Setup

732 [TDO KB](#) November 20, 2025 [Radiography](#) 25.44K

Overview

This article will cover how to use the CS 1200 and CS 1500 Intraoral Cameras to take images in TDO.

Detailed Instructions

1. Install the TWAIN driver for this device. If you don't have the installation files for the driver, you can find them [here](#) [External Link].

Note: TDO does not furnish device drivers nor assist in their installation. Contact Carestream Support if you need help installing the drivers for your device.

2. Launch Acquire in TDO.

3. Click on the 'Scan' button icon, this will bring up a list of TWAIN devices accessible by the computer.

4. Select the CS 1200 TWAIN or CS 1500 TWAIN (for CS 1500 intraoral camera) driver and click 'Select'. This will launch the CS 1200 software or CS 1500 software (for CS 1500 intraoral camera).

5. Acquire your image. (Only one image at a time can be sent to TDO)

6. In TDO's Acquire window, select a category or categories under which to save your image, then click 'Save'.

Online URL: <https://kb.tdo4endo.com/article.php?id=732>