


## Refresh buttons in Organizer

773 [TDO KB](#) October 2, 2023 [Organizer](#), [Organizer](#) 4.23K

In previous versions of TDO 12, Image Organizer had 3 different refresh buttons. In TDO 12.351d and above they are grouped into a dropdown.

Previous versions of TDO:

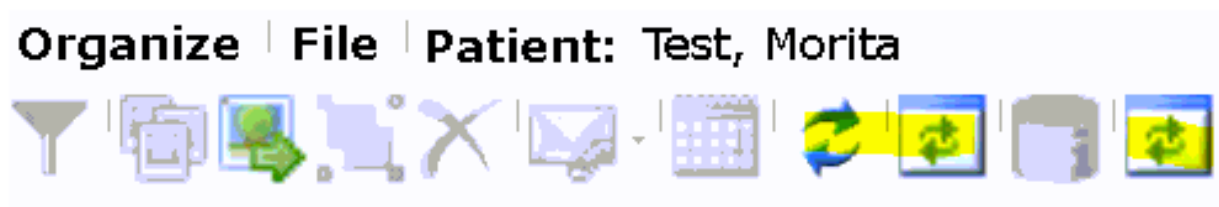

TDO version 12.351d and newer:




Note: In version 12.364n and newer, each button will have a tooltip describing its function that will display if you hover the mouse over the button.

### Steps:

- Open Image Organizer
- Click on the Refresh Dropdown Menu
- Select one of the categories:
  - Refresh Images: Reload images from xrays folder.
  - Recreate Thumbnails: Recreate thumbnails from original image.
  - Create Missing Burned Images: Create any missing image for copy and paste and/or drag and drop.

Note: 'Burned Image' refers to an image that has all annotations which have been added included.

The same options are available when right-clicking in the body of the Image Organizer. Simply right-click anywhere in the right pane of the Image Organizer.

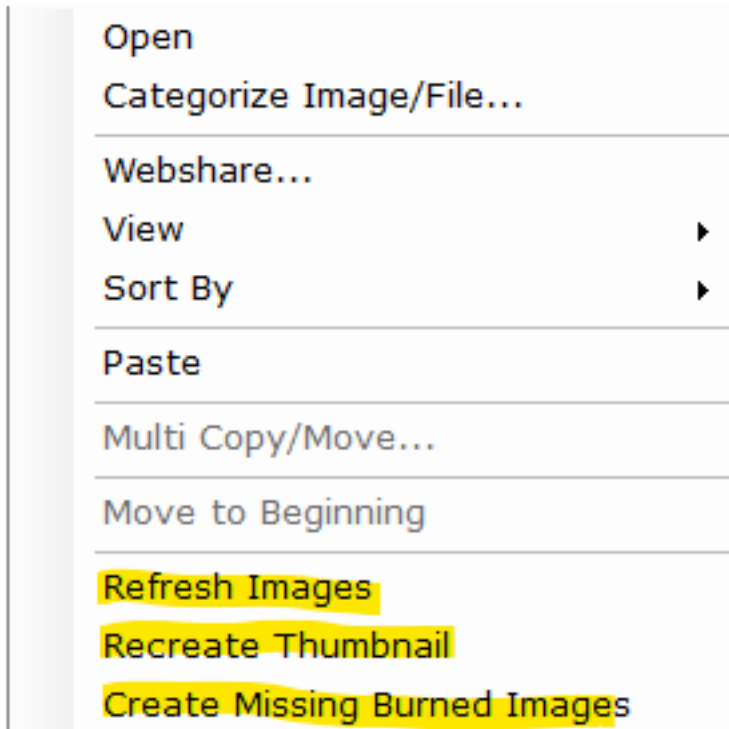

Online URL: <https://kb.tdo4endo.com/article.php?id=773>