

Carestream - 6100/6200 - taking continuous x-rays

862 [TDOKB1](#) November 21, 2025 [Acquire](#), [Acquire](#), [CBCT](#) 7.82K

Overview

This feature is designed to simplify the process of taking multiple X-rays with one click. CS 6100 and CS 6200 sensors work with this new functionality.

This feature is available in **12.403** and newer.


Detailed Instructions

- In Acquire page, click the Tooth button
- Check the “Continuous X Ray” box located between ‘Auto Stamp’ box and ‘Lectures’ button. Note: TDO will remember the selection next time you take an X-ray. If you don’t wish to take multiple X-rays you can simply uncheck the “Continuous X Ray” box.



The Carestream RVG Acquisition window will come up on the screen. NOTE: A timer will show with CS6100


6100

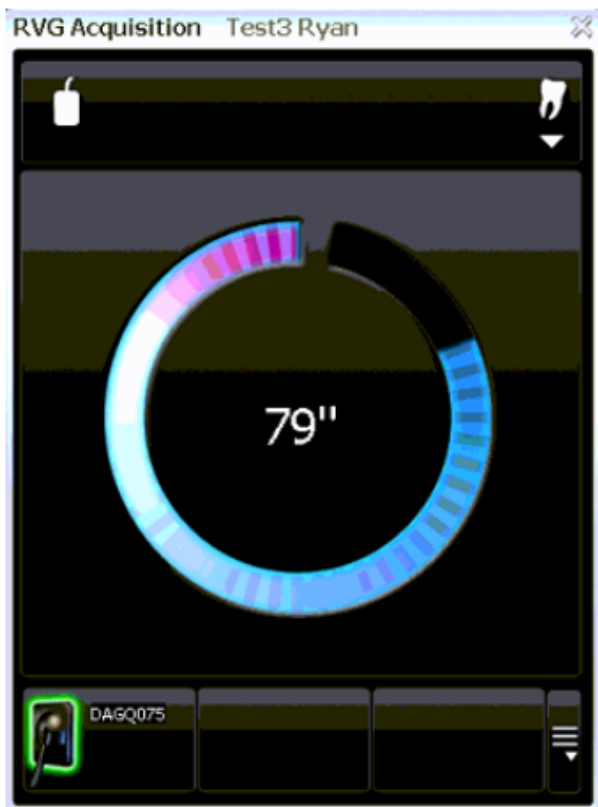


6200



- Take the first image.
- The first image will show in Acquire.
- Then you will see the Carestream RVG Acquisition will show on the screen again for another xray.

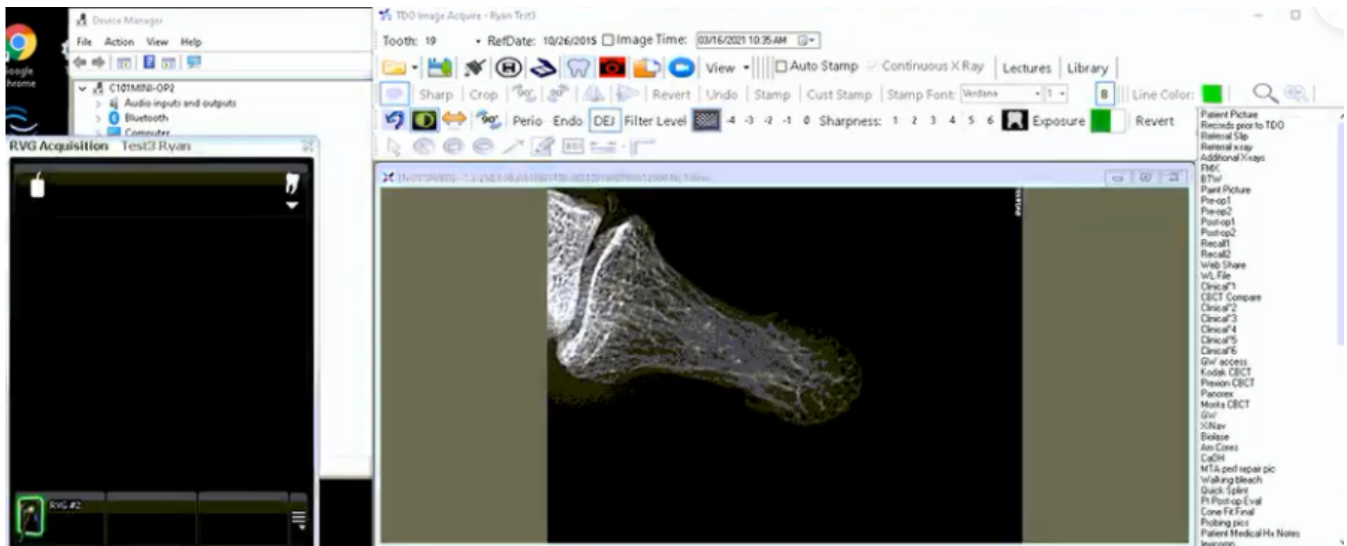

- The Carestream RVG Acquisition window will display as many times as you need.
Notes: Every time you take an X-ray, the X-ray will appear in Acquire before the Carestream RVG Acquisition window comes up on the screen, ready for another X-ray.
For CS6100 sensors: if the timer runs out of time, you will need to click the 'Tooth' button again to start the process.
- When done taking x-rays, you can simply click on the "X" button to close the Carestream RVG Acquisition window.

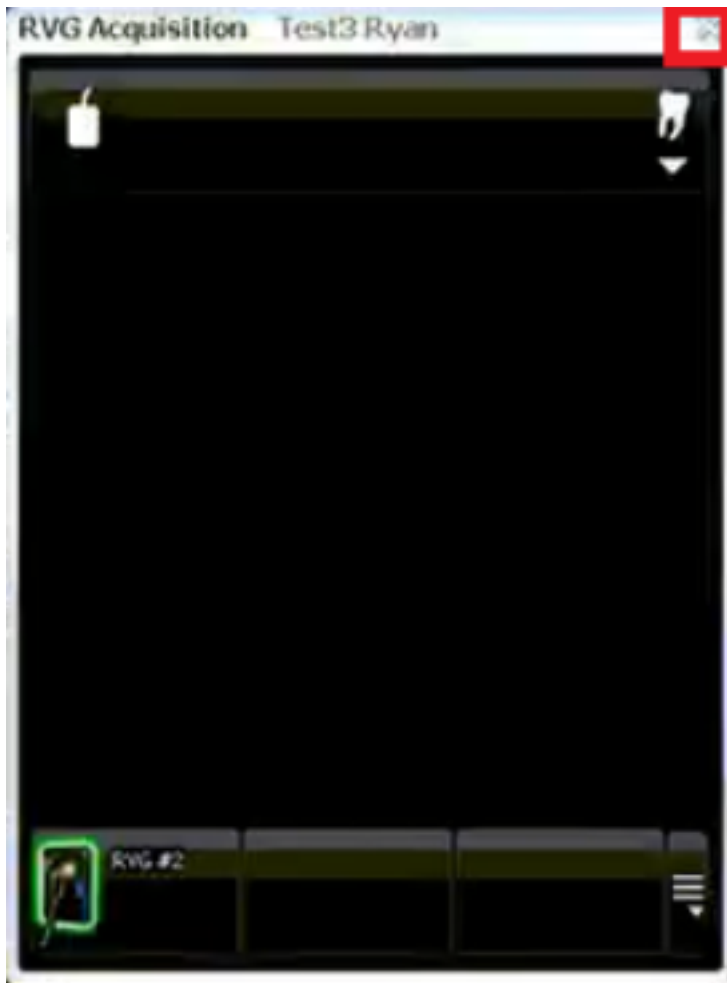

Online URL: <https://kb.tdo4endo.com/article.php?id=862>